

BONE EASY QUICKSMILE

FROM ZERO TO SMILE - IN JUST ONE
APPOINTMENT



PREVIOUSLY, THE PROCESS
OF DEFINING AND
VALIDATING THE FINAL SMILE
OF AN EDENTULOUS PATIENT
REQUIRED MULTIPLE
APPOINTMENTS AND
LENGTHY LAB WORK.

WHAT IF YOU COULD ACHIEVE
ALL REFERENCES FOR YOUR
IMPLANT PROCEDURE IN
JUST **ONE SESSION?**



MEET QUICKSMILE

AN INNOVATIVE KIT
THAT SIMPLIFIES AND
ACCELERATES THE
SMILE PLANNING
PROCESS



info@boneeasy.com

+351 929 013 061



WORKFLOW STEPS**1 ESTHETIC SELECTION**

**PATIENT SELECTS
THE SHAPE THEY
PREFER**

**2 DIMENSION SELECTION**

DENTISTS SELECT THE BEST ARCH SIZE



WORKFLOW STEPS

3 TRAY SELECTION

PICK THE CORRESPONDING
SCHRNEIDERMAKER TRAY



4 WAX ADDITION

APPLY **RED** WAX TO DEFINE THE HEIGHT



5 TEETH POSITIONING

PUSH THE CHOSEN TEETH INTO THE WAX



WORKFLOW STEPS

6 SILICONE

**APPLY SILICONE AND SEAT
IN MOUTH**



7 ADJUSTMENTS

**ADJUST THE MEDIAL LINE
AND THE OCCLUSION
PLANE**



WORKFLOW STEPS

8 PATIENT APPROVAL

TAKE A PICTURE TO SHOW
THE PATIENT THE CHOOSSEN
SMILE



9 CBCT SCAN

PATIENT UNDERGOES A
CBCT WITH THE DEVICE ON



10 DIGITAL PLANNING



WHY IS THIS THE BETTER ALTERNATIVE?

- NO WAITING. ALL-IN-ONE APPOINTMENT
- THE PATIENT SELECTS & APPROVES EVERYTHING
- RECEIVE ALL DIGITAL REFERENCES INSTANTLY



TRADITIONAL TO QUICKSMILE

- | | | |
|-------------------------|---|-----------------------|
| • DURATION 1 WEEK | → | 1 APPOINTMENT |
| • MULTIPLE APPOINTMENTS | → | SINGLE VISIT |
| • LAB DEPENDENCE | → | IN CLINIC- REFERENCES |
| • ADJUSTMENTS OVER TIME | → | REAL TIME ADJUSTMENTS |