

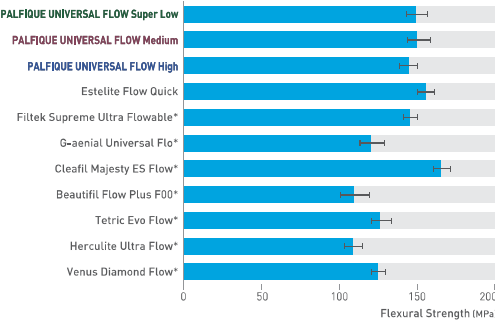


Excellent physical properties

Flexural Strength

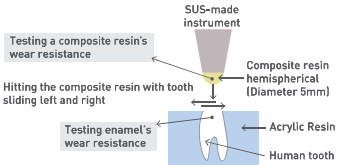

Wear resistance

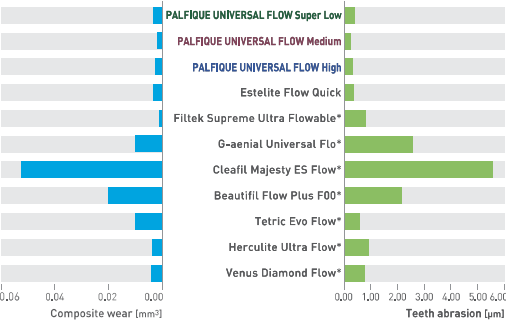

Product Packages Available

PALFIQUE UNIVERSAL FLOW - Resin-based Dental Restorative Material -

Super Low L-SYR
Contents: 1-1,8ml (3,0g) syringe, 10-tips
Available in 6 shades: A1, A2, A3, A3.5, A4, A5

Medium L-SYR
Contents: 1-1,8ml (3,0g) syringe, 10-tips
MEDIUM SYRINGE
Contents: 1-1,0ml (1,7g) syringe, 6-tips
Available in 12 shades: CE, BW, A1, A2, A3, A3.5, A4, A5, B3, 0PA2, 0PA3, 0PA4

High L-SYR
Contents: 1-1,8ml (3,0g) syringe, 10-tips
Available in 7 shades: A1, A2, A3, A3.5, 0PA2, 0PA3, 0PA4

Accessory - TOKUYAMA Dispensing Tip / Contents: 50-tips - TOKUYAMA Plastic Tip / Contents: 50-tips - PALFIQUE UNIVERSAL FLOW Shade Guide

Related Product

PALFIQUE UNIVERSAL BOND - Self-cured Dental Universal Adhesive -

KIT
- PALFIQUE UNIVERSAL BOND BOND A / 5mL x 1
- PALFIQUE UNIVERSAL BOND BOND B / 5mL x 1
- Disposable Applicator (Fine) x 25
- Mixing Well x 1
- Disposable Mixing Well x 5

Refill
- PALFIQUE UNIVERSAL BOND BOND A / 5mL
- PALFIQUE UNIVERSAL BOND BOND B / 5mL
- Disposable Mixing Well / 50

Specifications are subject to change without notice based on Manufacturers discretion.

Manufactured by
Tokuyama Dental Corporation

38-9, Taitou 1-chome, Taitou-ku, Tokyo 110-0016, Japan

TEL: +81-3-3835-2261 FAX: +81-3-3835-2265

Email: export-2@tokuyama-dental.co.jp

URL: www.tokuyama-dental.com



PALFIQUE UNIVERSAL FLOW

RESIN-BASED DENTAL RESTORATIVE MATERIAL



Excellent esthetics

Wide and natural color matching

PALFIQUE UNIVERSAL FLOW is basically designed to make it possible that most cases can be restored by limited mono shade thorough its excellent shade matching/color blending effect.




The color compatibility of PALFIQUE UNIVERSAL FLOW A2 shade when it is filled in each artificial tooth (B1, A1, B2, A2, C1, A3) after creating a cavity 4 mm in diameter and 2 mm deep. By using black light, we can see the fluorescence of PALFIQUE UNIVERSAL FLOW where it was filled.




High Light Diffusion

PALFIQUE UNIVERSAL FLOW has the highest light diffusion by laser-transmission test. It can be possible in clinical to achieve wide color matching and the invisible margin.

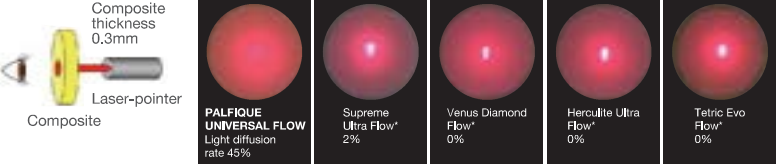


Filler components


Supra-nano spherical filler (200nm, silica-zirconia filler) and round shaped composite filler (include supra-nano spherical filler) are used as filler of PALFIQUE UNIVERSAL FLOW. The adoption of both filler can be contributes to the esthetic properties (wide color matching, excellent polishability) and the handling properties (easy extrusion, non-sticky).

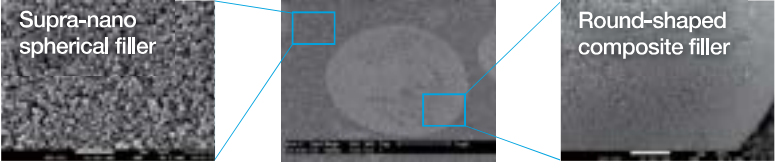

Polishability

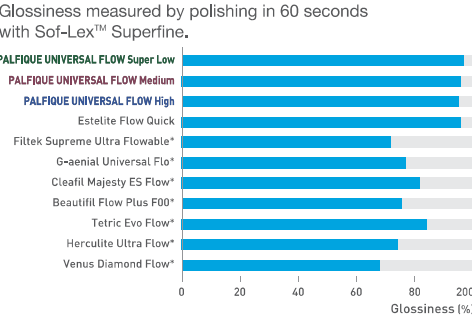

Gloss retention

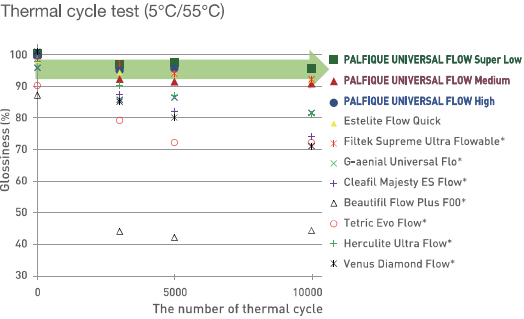

Three flowability options and excellent handling

Super Low	Medium	High
Non slumping Non running Precision stacking	High versatility Less slumping Less running	Easy placement Flow well
Flowability 	Flowability 	Flowability
Recommended Cavities 	Recommended Cavities 	Recommended Cavities
Preparation	Preoperation	Preoperation
Placement of PALFIQUE UNIVERSAL FLOW (Super Low A2)	Placement of PALFIQUE UNIVERSAL FLOW (High OPA2 + Medium CE)	Placement of PALFIQUE UNIVERSAL FLOW (High OPA2 + Medium CE)
Final Restoration	Final Restoration	Final Restoration
6 shades 	12 shades 	7 shades

※1) 0.1g 37°C stand for 2 minutes (horizontal) ※2) 0.03g 37°C stand for 1 minute (vertical)