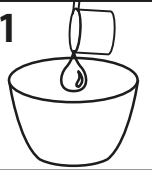
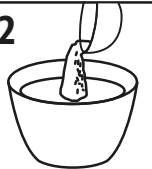
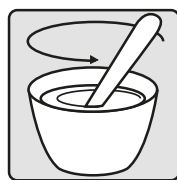
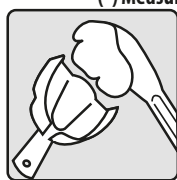
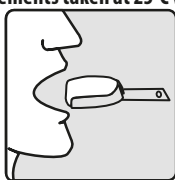
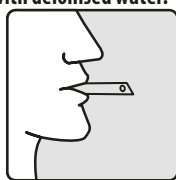
		20 ml. + 9 g. ≈ 1 x 		1 
		40 ml. + 18 g. ≈ 2 x 		2 

Type I
 (*) initial setting time **1'55"**
 (*) working time **1'45"**
 (*) mixing time **45"**
 (*) Measurements taken at 23°C with deionised water.

			
--	--	--	---

Type II
 (*) mixing time **45"**
 (*) working time **2'45"**
 (*) initial setting time **2'55"**
Only 30" in mouth

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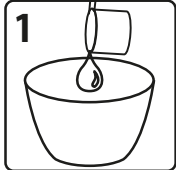
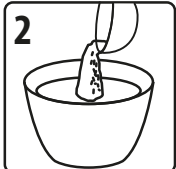
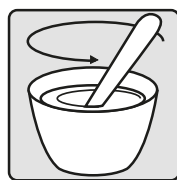
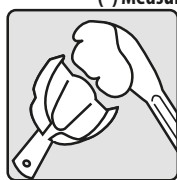
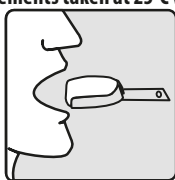
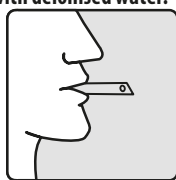
(*) working time **1'45"**

(*) mixing time **45"**

(*) Measurements taken at 23°C with deionised water.

Type II

(*) mixing time **45"**

(*) working time **2'45"**

(*) initial setting time **2'55"**

Only 30" in mouth

TYPE I: FAST SETTING

TYPE II: NORMAL SETTING

It does not require special treatment before use. Follow the powder/water ratio. Pour the powder into the water, mix vigorously. Work in compliance with the declared times (*), measurements were taken at 23°C with deionized water. When using specific mechanical mixers for alginate-based impression materials, follow the manufacturer's instructions. Variations in powder/water ratio, in temperature and/or water hardness and in the speed and/or friction during mixing cause changes in the working time and in the initial setting time. Wash, rinse, shake and disinfect the impression (specific alcohol-based spray disinfectants are recommended). By keeping the impression in a sealed plastic bag, without adding anything else, the model can be poured within 168 hours. Recommended plasters: Kromotipo 3, Singletipo 3, Kromotipo 4, Singletipo 4, Ortotipo.

WARNINGS: For use, wear face mask, protective gloves, eye protection and protective clothing. The powder is "Dust Free", however, caution is recommended to avoid inhalation. In case of contact with eyes wash thoroughly with water. After taking the powder close the bag immediately. Store in a cool and dry place. Use within the expiry date. Do not use on patients allergic to spearmint flavour. Any possible serious accident related to the product must be reported to Lascod and to the Competent Authority of the Member State in which the user is established.

Net weight: 450 g.
Net volume after mixing: 1250 ml.

20 ml. + 9 g.
 $\approx 1 \times$ $\approx 1 \times$

40 ml. + 18 g.
 $\approx 2 \times$ $\approx 2 \times$

1

2

Type I

(*) initial setting time **1'55"**

(*) working time **1'45"**

(*) mixing time **45"**

(*) Measurements taken at 23°C with deionised water.

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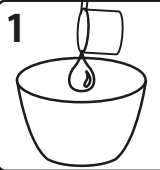
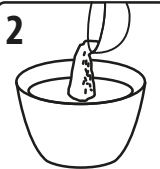
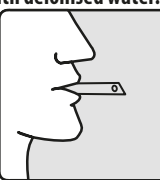
Poids net : 450 g.

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			
	20 ml. + 9 g. ≈ 1 x 	≈ 1 x 	
	40 ml. + 18 g. ≈ 2 x 	≈ 2 x 	
	(*) initial setting time 1'55" (*) working time 1'45" (*) mixing time 45"		
Type II	(*) Measurements taken at 23°C with deionised water.		
	(*) initial setting time 2'55" (*) working time 2'45" (*) mixing time 45"		

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