Manufacturer

SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

EC REP

SHOFU DENTAL GmbH
An der Pönt 70, 40885 Ratingen, Germany

DURA-POLISH

DURA-POLISH DIA

02.0:2020-09
72400

en

READ CAREFULLY BEFORE USE

de

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A LIRE ATTENTIVEMENT AVANT USAGE

nl

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DURA-POLISH is Al₂O₃ particle impregnated polishing wax for prepolishing and polishing.
DURA-POLISH DIA is a diamond particle impregnated polishing wax for super polishing.

Make DURA-POLISH or DURA-POLISH DIA adhere to brush or felt wheel with slow revolution speed and, by lightly pressing brush or felt wheel against the restorations, polishing can be easily done.

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Type	DURA-POLISH	DURA-POLISH DIA
Application	Hard resin, Metal, Denture resin	Hard resin, Porcelain
Effect	Prepolishing and finish-polishing	Super polishing for luster

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Cautions

- Use this product with less than 10,000 min⁻¹ revolution speed since high-speed revolution may cause spattering of the polishing wax.
- Do not strongly press when adhering this product to brush, etc. in order to avoid splash out from the container.
- Use local exhaust fan and dust-proof.
- Wear eye-protecting glasses when operating.
- Use this product for indicated purpose only.

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Lieferform

DURA-POLISH: Dose 20 g
DURA-POLISH DIA: Dose 5 g

Packaging

DURA-POLISH: jar 20 g
DURA-POLISH DIA: jar 5 g

Location of the symbols glossary:
https://www.shofu.com/en/symbols_glossary/

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Emballage

DURA-POLISH: pot de 20 g
DURA-POLISH DIA: pot de 5 g

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Verpakking

DURA-POLISH: pot 20 g
DURA-POLISH DIA: pot 5 g

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Envasado

DURA-POLISH: frasco 20 gr.
DURA-POLISH DIA: frasco 5 gr.

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